

ICONOCLAST® BOOT MEASUREMENT CHART

~ Measurements are good for both the Orthopedic Sport Boots & the Orthopedic/Rehabilitation Boots.

To ensure you are using the proper size boots for your horse please take a measurement of your horse's fetlock. (See the picture below)

We recommend using a soft tape measure. If you do not have a soft tape measure you can use something like string, wrap it around the fetlock, mark where the end meets the rest of the string and then simply lay it flat next to a tape measure, ruler or yard stick to get your measurement.



Small: 9 1/2" and Under
Medium: 9 3/4" to 10 1/2"
Large: 10 3/4" to 11 1/2"
X Large: 11 3/4" and Larger

To ensure proper application of your Iconoclast® Boots please refer to the instructions on the packaging. You can also log on to www.westernlegacysales.com to watch our "Application Instruction Video" found under the Iconoclast® Tab on the home page.

If you require further instructions please contact your Iconoclast® dealer or email ward@westernlegacysales.com